



ReGrasp

Electrode Positioning Kit V2

USER MANUAL

HS-IFU-EPK-EN
Revision C
2023-12
Rx Only (US Only)

Table of Contents

Introduction	3
Device Description	3
List of Symbols	5
Safety Information	6
Kit Components	10
User Interface	11
Using the Electrode Positioning Kit	12
EPK General Operation	12
Electrode Positioning Kit Setup	13
Finding the Optimal Motor Points	15
EPK Non Garment Use	18
Transferring the Electrode Position to the GARMENT	20
(Optional) GARMENT FILM LINER Placement	22
Fitting the GARMENT	24
Cleaning and Disposal	25
Contact and Reordering Information	26

Introduction

Device Description



The Electrode Positioning Kit is an accessory to the ReGrasp electrical stimulation system for use by clinicians. Its intended purpose is to:

- Identify optimal motor points on the forearm
 - The point where the nerve meets the muscle differs in each muscle and in each person. The Electrode Positioning Kit allows clinicians the ability to identify what are known as the optimal motor points, or where to stimulate the muscle to render the best contraction of the wrist and finger joints.
- Identification of ideal candidates for the ReGrasp
 - Neurological diseases present in a variety of different ways. Symptoms can lead to damage in the nerves connected to muscle but are generally unknown until externally stimulated.









- Facilitates in-clinic full use of the ReGrasp without the requirement of a Garment. This is achieved by using stick on gel electrodes.



CAUTION: The Electrode Positioning Kit is intended for use by clinicians only.

List of Symbols

The following table lists the symbols that are used on the labels and in this manual:

Symbol	Definition	Application to the ReGrasp System
	Warning	Important information or instructions that should be observed to avoid potential injury or damage to the device.
	Follow Instructions for Use	The instructions for use contain critical instructions necessary to use the device safely.
	Model Reference (Catalogue/Re-order Number)	Reference the accompanying model number next to this symbol when re-ordering components.
	Lot/Batch Number	Reference the accompanying batch number next to this symbol if contacting support.
	Medical Device	The ReJoyce is a medical device.
	Type BF Applied Part	Components that make patient contact have a degree of protection against electric shock.
	Waste Electrical and Electronic Equipment (WEEE)	Do not dispose electronics with regular household waste. Consult local waste management authorities to determine the acceptable means to dispose of electronic equipment.
	Arm/Disarm	A button on the STIM that toggles arming and disarming of the stimulator.

Safety Information

Contraindications

- Do not use the Electrode Positioning Kit if a pacemaker or any other critical medical electrical equipment is in use. There is a remote chance that stimulation may interfere with such devices.
- Do not use the Electrode Positioning Kit if fractures or dislocations of the fingers, wrist or elbow are present. Stimulation with these conditions may complicate the existing injury.
- Do not use the Electrode Positioning Kit on areas where a cancerous lesion is present or suspected. Electrical stimulation over cancerous lesions may complicate the condition.

Warnings

- Never attempt to use the Electrode Positioning Kit to stimulate parts of the body other than the hand and forearm. Stimulation to other areas such as the head, neck or torso can result in serious injury or death.
- Stimulation should not be applied over the neck or mouth. Doing so may cause contractions that affect breathing and/or heart-rate. Severe spasm of the laryngeal and pharyngeal muscles (muscles located in the neck and mouth) may occur and the contractions may be strong enough to close the airway or cause difficulty in breathing.
- Stimulation should not be applied transthoracically (across the chest) or upper back, in that the introduction of electrical current into the heart may cause cardiac arrhythmias (irregular heart-beats).
- Stimulation should not be applied transcerebrally (across the head), or directly on the eyes. Doing so may cause nausea, headache, or dizziness.
- Do not use the Electrode Positioning Kit on, or close to injured, swollen, infected, or inflamed areas or skin eruptions (e.g. phlebitis,

thrombophlebitis, varicose veins, etc.). Doing so may complicate the existing condition or prolong healing.

- Interference is possible between the ReGrasp and other electrical equipment.
- Do not use the device with any surgical equipment, or while near diagnostic equipment such as MRI, CT and ultrasound machines. Doing so may interfere with the equipment, result in burns, or produce device instability.
- Do not use the device within 1 meter of any shortwave or microwave therapy equipment, as this may affect the performance of the ReGrasp.
- Do not immerse the **STIM**, **PUCK**, or **PLACEMENT DOCK** in water. Doing so may result in electrical shock and damage to the device.
- Never attempt to disassemble, modify or repair any components of the system. Doing so may result in electrical shock and damage to the device.
- Do not use if any component is visibly damaged. Wires, connectors, and enclosures should be inspected for defects, breaks or obstructions before every use.
- The Electrode Positioning Kit is to be used only with the **GARMENT FILM LINER**, **ELECTRODES**, and **PLACEMENT DOCK** components supplied by Rehabtronics Inc.
- The **COTTON ELECTRODES**, **GEL ELECTRODES**, **GARMENT FILM LINER**, and **MARKING PEN** are intended for single use. Do not transfer between patients, and discard after use.

Precautions

- The Electrode Positioning Kit is designed to be used exclusively by a clinician on a patient in a controlled setting where no other activity will occur.

- Safety of powered muscle stimulators for use during pregnancy has not been established.
- Caution should be used for users with suspected or diagnosed heart problems, epilepsy or osteoporosis.
- Caution should be used over areas of the skin which lack normal sensation.
- Some patients may experience skin irritation or hypersensitivity due to electrical stimulation. The irritation can usually be reduced by using an alternate electrode placement or by changing the stimulation parameters. If problems persist, discontinue use.
- Do not store or operate the Electrode Positioning Kit in conditions outside those listed in the Technical Information section of the ReGrasp User Manual. Doing so can damage components.
- Electrical components of the Electrode Positioning Kit require special precautions for electromagnetic compatibility and immunity. Refer to Electromagnetic Compatibility tables in the Technical Information section of the ReGrasp User Manual for details.
- Powered muscle stimulators should be used only with the leads and electrodes recommended for use by the manufacturer.
- Caution should be used in the presence of the following:
 - When there is a tendency to hemorrhage (external or internal bleeding) following acute trauma or fracture.
 - Following recent surgical procedures when muscle contraction may disrupt the healing process.
 - Local insufficiency (insufficient blood flow).
 - Occlusion (a blood flow blockage).
 - Arteriovenous fistula (an abnormal connection between an artery and vein) for the purpose of hemodialysis.

- Primary disorder of the vasculature (a disease of the arteries, veins, and lymphatics).
- A bone deformity in the area to be stimulated.
- Always remove the wettable COTTON ELECTRODE from the PUCK prior to wetting the electrode.
- Excess body hair where the Electrode Positioning Kit ELECTRODES touch may reduce electrode contact with the skin. If necessary, remove excess body hair prior to applying electrodes.
- Do not use the Electrode Positioning Kit without the ELECTRODES.

Adverse Reactions

- Skin irritation and burns beneath the ELECTRODES have been reported with the use of powered muscle stimulators.
- In the unlikely event that any of the following occurs, stop using your Electrode Positioning Kit immediately:
 - Signs of significant irritation or pressure sores where the Electrode Positioning Kit components contact the skin.
 - A significant increase in muscle spasticity.
 - A feeling of heart-related stress during stimulation.
 - Swelling of the hand, wrist, or forearm.
 - Any other unanticipated reaction.

User Interface

When the ReGrasp system **STIM** is placed into the **PLACEMENT DOCK** and powered-on, it will enter an electrode positioning mode:



1

Arm/Disarm Button

When armed, **PUCK** will deliver stimulation when its button is pressed.

2

Battery Indicator

Indicates STIM battery charge.

3

Conductivity Quality Number

Value	Meaning
6-10	Optimal Quality
3-5	Minimal Quality
0-2	Poor Quality or No Stimulation
*	STIM is not armed

Poor Quality Number may be due to:

- **PUCK** button is not pressed
- **GEL ELECTRODE** not applied properly
- **COTTON ELECTRODE** is not sufficiently wet
- **COTTON ELECTRODE** is not making good contact with patient's skin

4

Intensity Indicator

Indicates **PUCK** stimulation intensity.

Use the  and  buttons to adjust.

5

On/Off Button

Press and hold to turn **STIM** On/Off.

Using the Electrode Positioning Kit

EPK General Operation



1. Ensure that the **STIM** is removed from the **PLACEMENT DOCK** *whenever* electrodes are being attached to or detached from the puck or electrode holders. This will prevent any unintended exposure to the stimulation.
2. Ensure that the switch is appropriately set for the mode of operation. The toggle switch on the Dock selects EPK Electrode Positioning mode or ReGrasp Mode.
3. When any of the electrode lead holders are not in use ensure that the protective caps are installed. During EPK mode the stimulation signal will also be active on the Open (Marked O) electrode as well. The protective caps will prevent inadvertent exposure to the stimulation signal.
4. We recommend that protective gloves are worn by the clinicians while using the EPK system
5. If it is challenging to setup the thumb electrode for a patient, omit the thumb electrode and use the system with the thumb intensity set to zero.

Electrode Positioning Kit Setup



Figure 1: Attach Wetted Cotton Electrode to Puck

1. Ensure that the **STIM** is removed from the **PLACEMENT DOCK**.
2. Use the Large Cotton Electrode for Hand Open and Hand close function. The Small cotton electrode is used for the Thumb location.
3. Thoroughly wet the **COTTON ELECTRODE**. Use only clean water. The cotton should be saturated. Squeeze **ELECTRODES** lightly to remove excess water. The **ELECTRODES** should be as wet as possible, without dripping water.
4. Snap the **COTTON ELECTRODE** onto the PUCK (Figure 1).
5. Snap the **GEL ELECTRODE** onto the black **NEUTRAL LEAD** (Figure 2).







Figure 2: Attach Gel Electrode to Neutral Lead

6. Stick the adhesive side of the **GEL ELECTRODE** onto the patient at the back of the forearm, just proximal to the ulnar styloid process - the bony part near the wrist (*Figure 3*).
7. With the **MARKING PEN** provided, draw around the **GEL ELECTRODE** on the patient's skin.
8. Ensure the Dock toggle switch is in the EPK position.



Figure 3: Gel Electrode Placement

9. Place the STIM onto the PLACEMENT DOCK and turn the STIM on by holding down the  button.
10. Arm the **STIM** by pressing the  button.
11. Set the intensity level by toggling  and  buttons on the **STIM**.



It is recommended to start at a low intensity (less than 10) for a patient exposed to FES for the first time.




Do not share **electrodes** between users. Discard electrodes after use.


Finding the Optimal Motor Points

NOTE: The Electrode Positioning Kit is intended to help find the optimal motor points on the patient's forearm. For thumb electrode positioning, refer to the ReGrasp system User Manual.

1. Place the wetted **COTTON ELECTRODE** attached to the **PUCK** against the muscles of the patient's forearm.
 - Place against the posterior aspect of the forearm (extensor muscles) to probe for hand-open response:

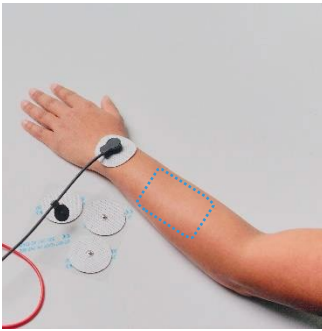

Left arm

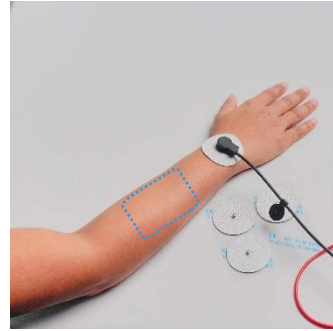

Right arm

Suggested Placement



Left arm



Right arm

Puck against posterior

- Place against the anterior aspect of the forearm (flexor muscles) to probe for hand-close response:

Left arm



Right Arm



Suggested
Placement Area



Puck against
anterior
forearm



Left arm

Right Arm

2. Press the button on the **PUCK** to start stimulating and observe the patient's hand response.



Only press the button when the electrode is firmly pressed on the desired location on the user's skin.

3. Probe for the optimal motor point by keeping the button pressed and moving the **PUCK** around on the surface of the patient's forearm and

observing where the strongest hand response is achieved (greatest movement in the fingers with minimal discomfort).

- Finding the right intensity and location may take time. Ensure firm contact is maintained while stimulating and the PUCK is in contact with skin.



Be sure to **release the button** on the **PUCK** *prior* to removing from the user's skin; this will reduce any stinging sensation to the user or inadvertent stimulation to yourself.



Figure 3: Mark optimum motor points

4. Once the optimal motor point is found, trace the PUCK's position on the patient's forearm using the MARKING PEN (Figure 4).
5. Repeat steps 1-4 on the other side of forearm, so that the optimal motor points for hand-open (posterior forearm) and hand-close (anterior forearm) are marked.



Avoid touching the user while stimulation is active. It is also recommended to wear a suitable medical gloves to reduce the probability of being exposed to the stimulation.



If a stronger contraction is desired and the user is comfortable with the existing stimulation levels, the stimulation intensity can be increased.

6. Remove the **STIM** from the **POSITIONING DOCK**. This will turn off any functionality of the unit.


EPK Non-Garment Use

The non-garment use allows clinicians to achieve full ReGrasp functionality on patients without the use of a Garment.

Once the locations of the electrodes have been identified with the Puck, the supplied gel electrodes can be placed on the patient at each of the located motor points. Snap the Gel electrode onto the electrode holders before placing on the patient as attaching the electrode holder to an electrode already on the patient is more difficult. The location of the Reference electrode remains the same as when used in EPK placement mode by connecting it to the Black electrode lead holder.


Three other gel electrodes are then connected to the 3 red colored stimulation output channels that are marked as follows:

- O – (Open Hand Channel) – Use Large Electrode
- C – (Close Hand Channel) – Use Large Electrode
- T - (Thumb Channel) – Use Small Electrode

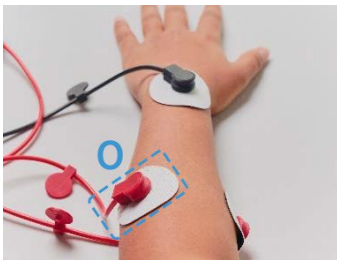


Anterior view - Left arm

Suggested
Placement
Area:
O – Open
Hand
Channel

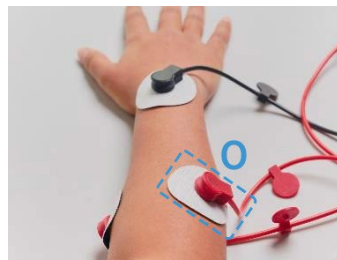

Anterior view - Right arm

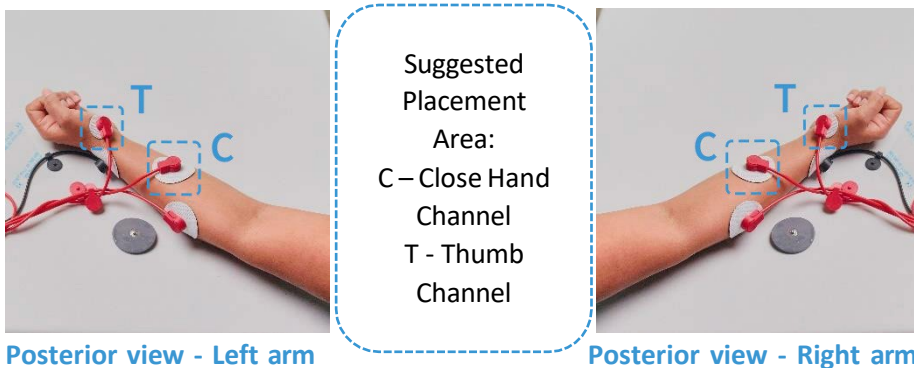

Once the electrodes are in position toggle the switch on the EPK dock from EPK mode to ReGrasp mode. In ReGrasp mode the ReGrasp STIM behaves the same way as it does when using a garment.

To start out set each of the intensities of the channels Open, Close and Thumb to the appropriate levels. Observe the patients muscle movement and ask the patient for feedback as the intensities are increased.

All modes of the ReGrasp are available in this configuration which includes:

- FES (Functional Electrical Stimulation)
- Exercise mode which includes both Strength and Cardio functions

For FES operation the HMS can function normally by means of head motions but the HMS can also be placed in the HMS trigger button dock. This will allow the cycling of the FES cycle by pressing the button. The cycle is hand open, hand close and then relax (no stimulation). The button is a convenient way to cycle through FES mode as it can be operated by the clinician or the patient.

Note that while the ReGrasp STIM is seated in the EPK dock we do not recommend using the Tap function to trigger the FES cycling.

Transferring the Electrode Position to the GARMENT

1. Place the **TRANSFER SHEET** onto the patient's forearm (*Figure 6*).
 - Make sure the thumb image on the **TRANSFER SHEET** is on the same side as the patient's thumb.
 - Line up the circle on the **TRANSFER SHEET** where the **GEL ELECTRODE** was marked on the patient's wrist.
2. Find the marked motor points on the patient's arm, and trace the outlines onto the **TRANSFER SHEET** (*Figure 5*).
 - Be sure not to shift the **TRANSFER SHEET** at all when copying the marked points.

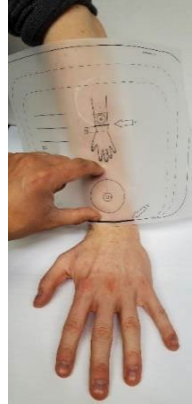



Figure 5: Place **TRANSFER SHEET**



Figure 4: Trace marked motor points

NOTE: If TRANSFER SHEET is too short to copy the marked motor points, you can move the TRANSFER SHEET up the arm, but the distance must be recorded to properly fit the system GARMENT.

Once the TRANSFER SHEET is positioned so that the marked motor points can be traced, measure the distance between thick bold line on the TRANSFER SHEET and the wrist joint, and record this distance as X on the TRANSFER SHEET.

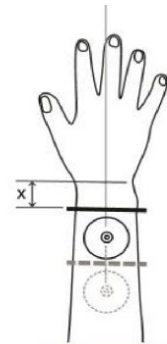

3. Remove the **TRANSFER SHEET** and clean the marks off the patient's arm.
4. Place the **TRANSFER SHEET** onto the inner aspect of the ReGrasp system **GARMENT** with the circle outline matched up with the **REFERENCE** electrode of the ReGrasp **GARMENT** (*Figure 7*).

- Be sure to *flip* the **TRANSFER SHEET** over. If you miss this step, the electrodes will sit on the wrong side of the **GARMENT**.

5. Move the velcro for the **GARMENT**'s HAND-OPEN and HAND-CLOSE electrodes so that their positions correspond to the traced positions on the **TRANSFER SHEET** (*Figure 8*).

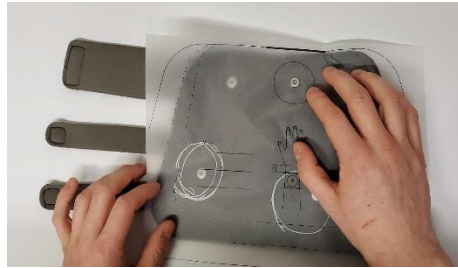


Figure 6: Line up TRANSFER SHEET with ReGrasp GARMENT



Figure 7: Move GARMENT electrode sockets

(Optional) GARMENT FILM LINER Placement

If increased sanitary protections are desired, an optional **GARMENT FILM LINER** can be applied to the ReGrasp **GARMENT** prior to donning on the patient as follows:

1. Remove the paper side of the **GARMENT FILM LINER**, ensuring that the adhesive side of the liner does not fold in on itself.
2. Place the **GARMENT FILM LINER** on the ReGrasp system **GARMENT**, with the adhesive side down so that it sticks to the inner aspect of the **GARMENT**.
3. Using a sharp-edged tool, cut the areas of the **GARMENT FILM LINER** that sits on top of the electrode sockets.
4. **Remove the shiny layer** of the **GARMENT FILM LINER**. This layer is rather slippery and will chafe if left on the **GARMENT** when worn.
5. Once the cutout pieces of the liner have been removed over the electrode sockets, reapply the **GARMENT**'s electrodes to their respective positions.



The GARMENT FILM LINER is not to be relied upon completely for protection from communicable disease. Consider disinfection or disposing of glove if patient has communicable disease or after extended use.



Cutting a liner can cause cuts to the clinician. Wear cut-resistant gloves or use a knife with extreme caution.

Fitting the GARMENT

1. After the ReGrasp **GARMENT** electrodes have been positioned according to the markings on the **TRANSFER SHEET**, fit the **GARMENT** on the patient according to the instructions in the ReGrasp system User Manual.

- Ensure that the **GARMENT**'s **REFERENCE** electrode is in the same position as the **GEL ELECTRODE** was during use of the Electrode Positioning Kit (*Figure 10*).
- NOTE: if a distance, X , was recorded on the **TRANSFER SHEET** during the "Transferring the Electrode Position to the **GARMENT**" step, the edge of the **GARMENT** should be this distance from the wrist joint (*Figure 9*).



Figure 9: Default Garment Position

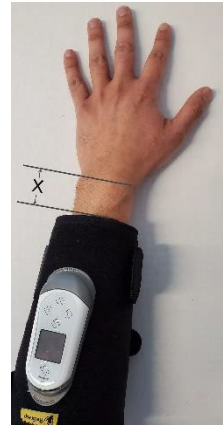


Figure 8: Adjusted Garment Position

2. Place the **STIM** into the **GARMENT** and follow the instructions in the ReGrasp system *User Manual* to set the intensities to each electrode.
 - Check for any differences between the initial set up.
 - Intensity and location may differ slightly between the two so a little bit of adjustment may be necessary.

Cleaning and Disposal

- Plastic parts of the Electrode Positioning Kit may be cleaned by wiping with a wet cloth, or low-level disinfected using Ethanol or Isopropyl alcohol. Allow to dry completely before reusing.



Do not immerse any components in water.

- The **COTTON ELECTRODE**, **GEL ELECTRODE**, **TRANSFER SHEET**, **GARMENT FILM LINER**, and **MARKING PEN** are intended for single-patient use, and should be disposed of after use. These components can be safely disposed with local municipal waste.



Do not attempt to wash or reuse these components. Doing so may result in skin irritation, infection, or have unknown effects on performance.

Contact and Reordering Information

For device support, or to reorder components, contact Rehabtronics Inc. via any of the following channels:



Web: www.rehabtronics.com

E-Mail: support@rehabtronics.com

Toll Free (US & Canada): [1-800-481-3214](tel:1-800-481-3214)

International Phone: (+1) 780-701-5167

For inquires or reordering outside Canada, contact your local ReGrasp distributor.