






### 1. Charge

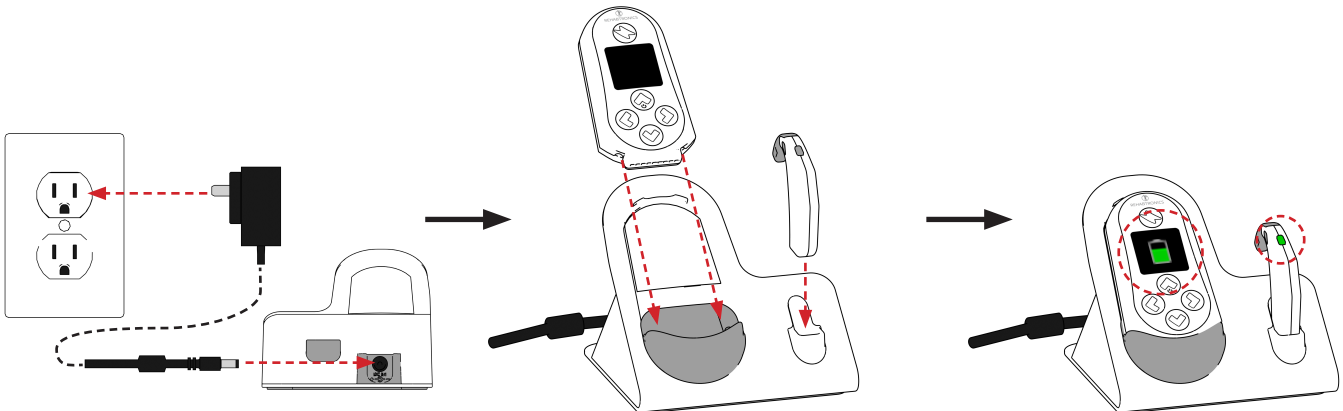


### 2. Wet and Attach Electrodes

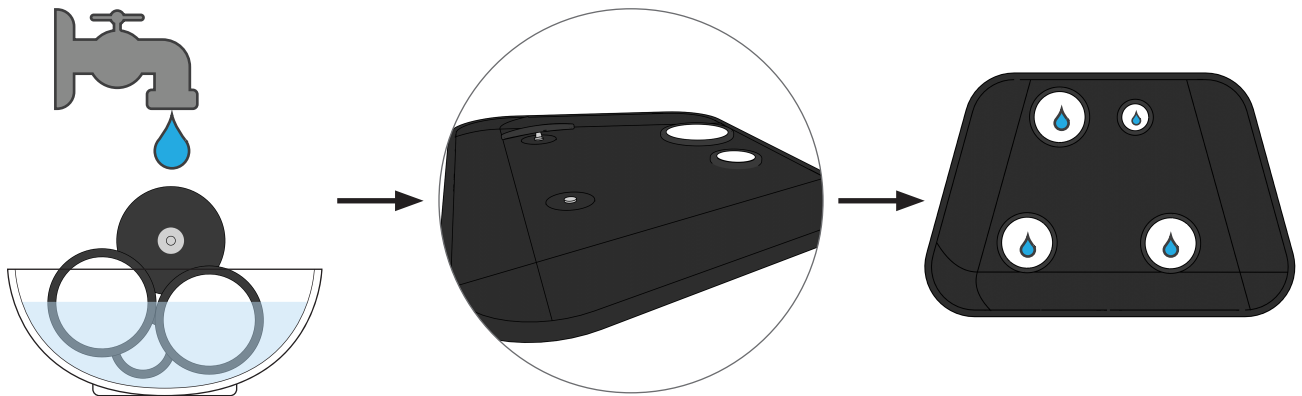


### 3. Don Garment

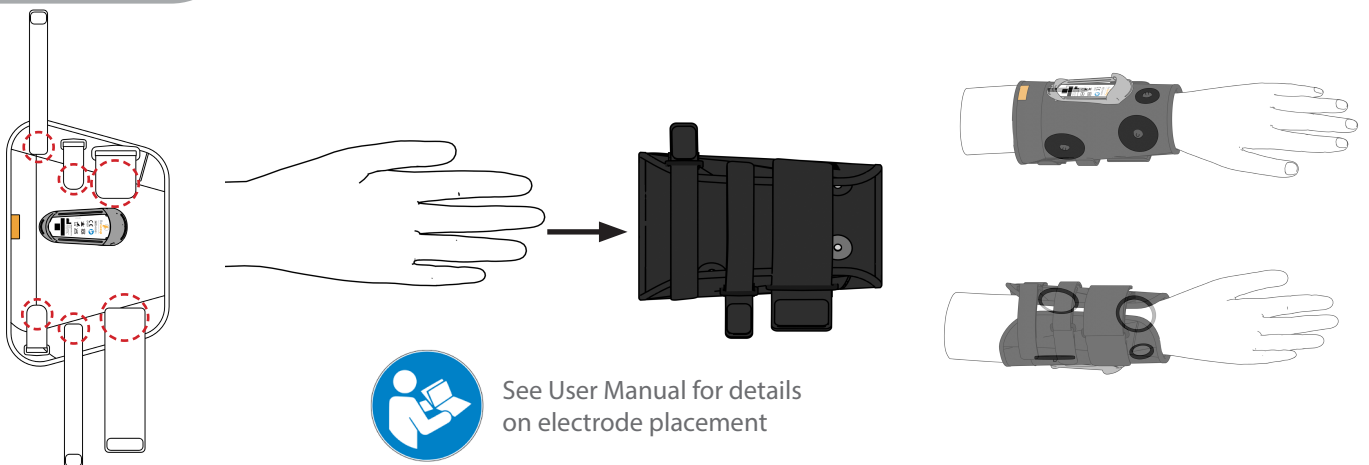

### 4. Connect Stim and Turn On

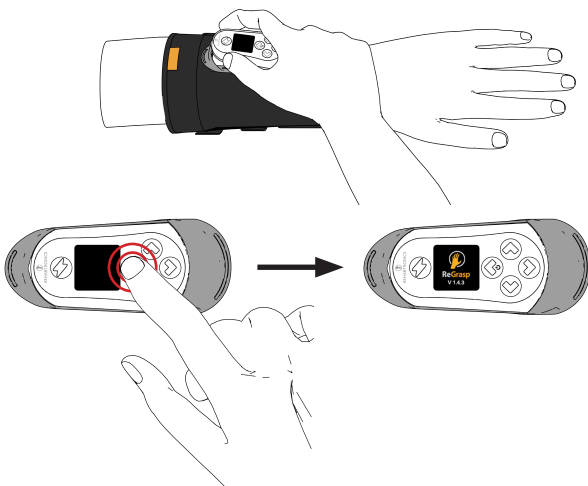


### 5. Enable STIM (⚡)

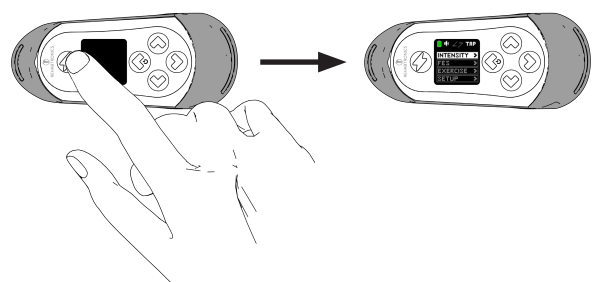





### 6. Set "Hand Open" Intensity

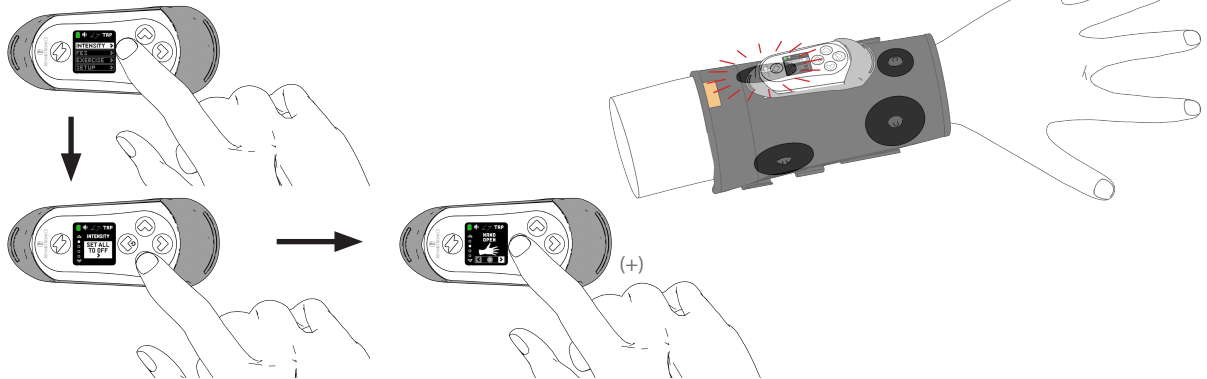


### 7. Set "Hand Close" Intensity

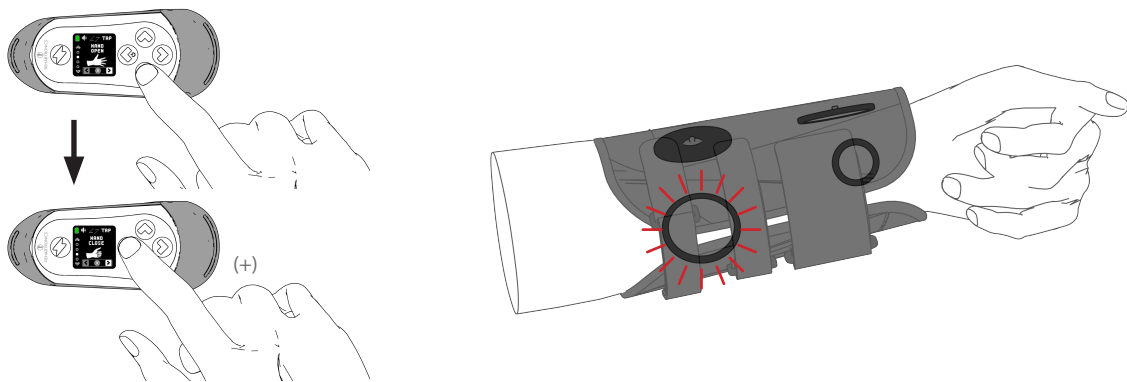

### 8. Initiate "FES" Mode

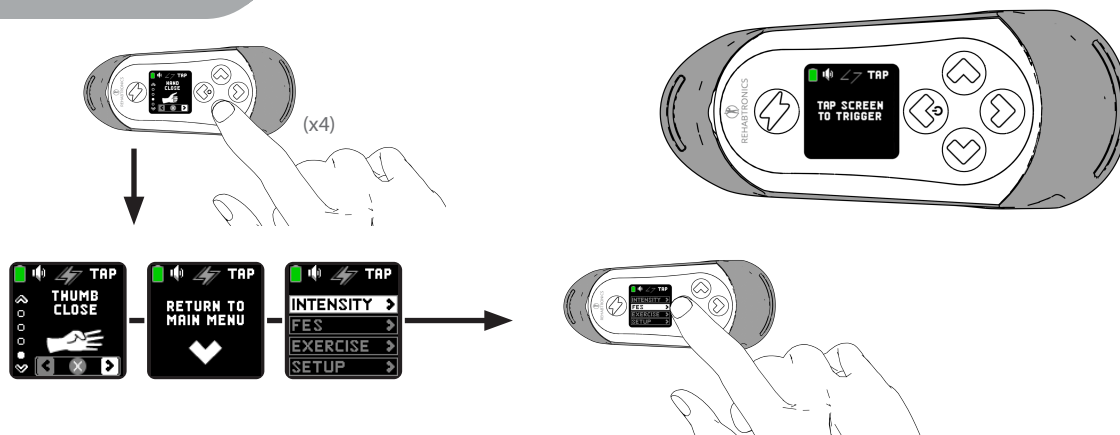

### 9. Tap Stim Display to Cycle Stimulation

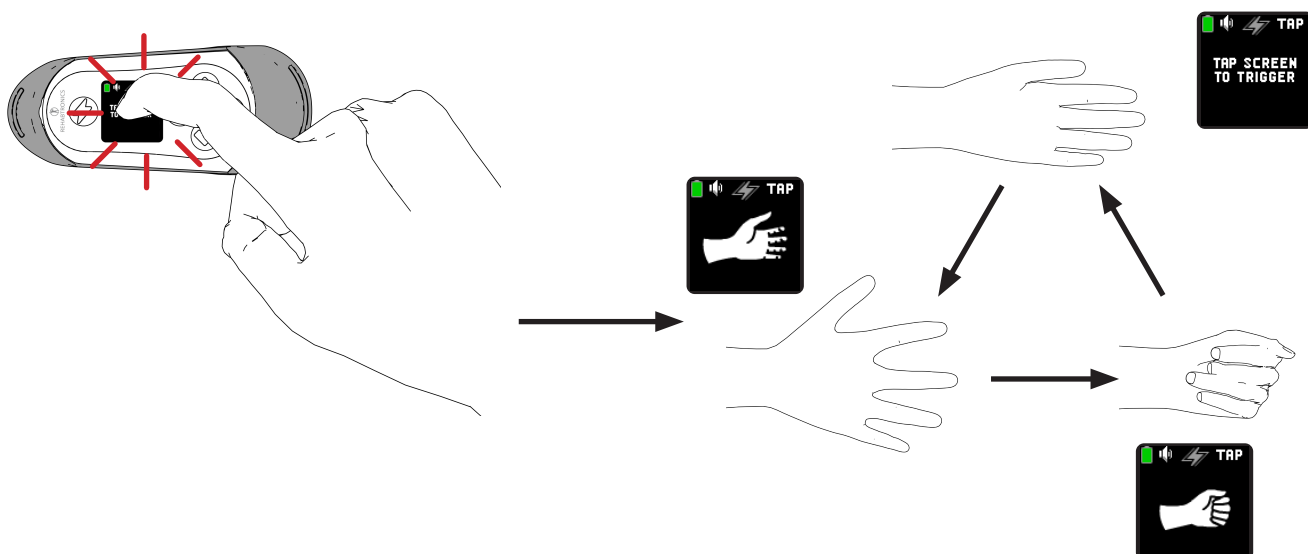




# ReGrasp

## Quick Start Guide - Right Hand




Refer to User Instructions


### 1. Charge

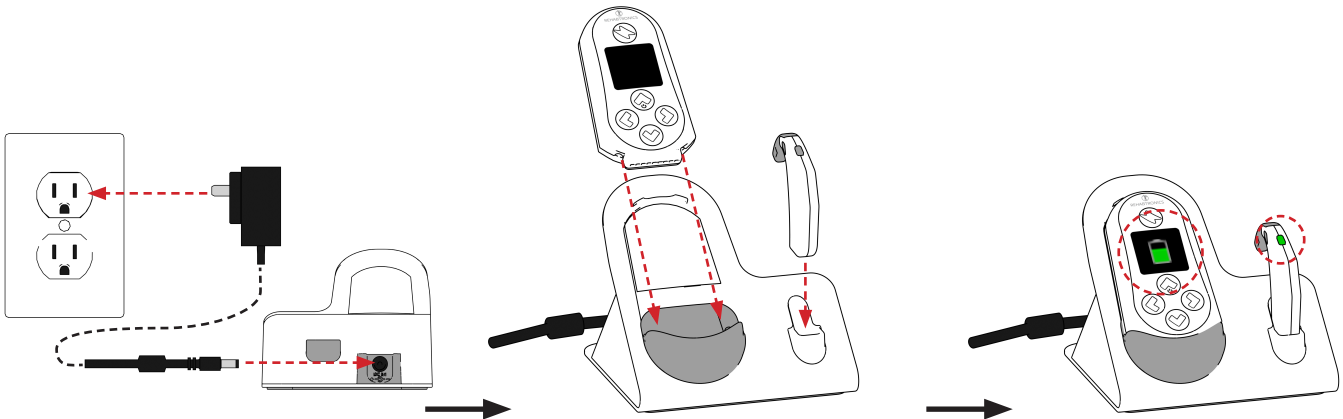


### 2. Wet and Attach Electrodes

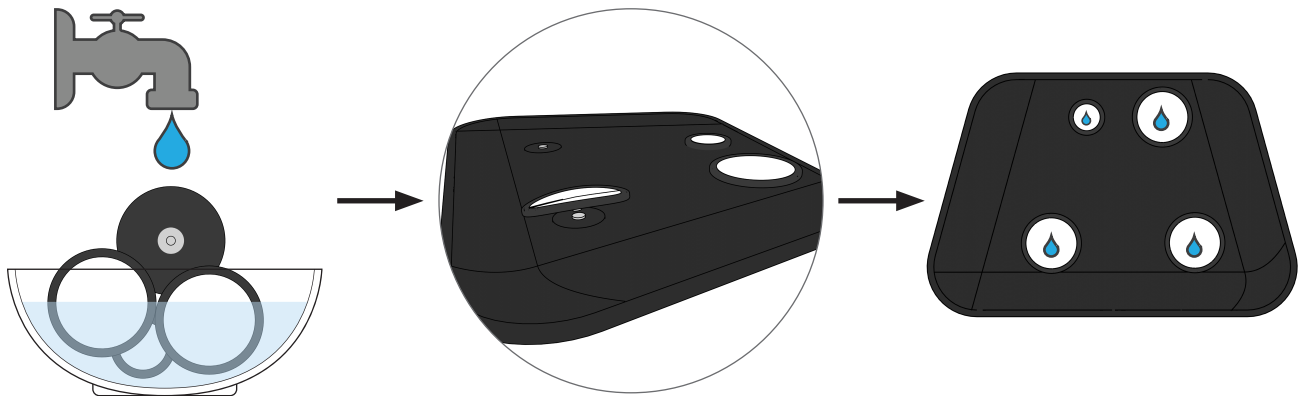


### 3. Don Garment

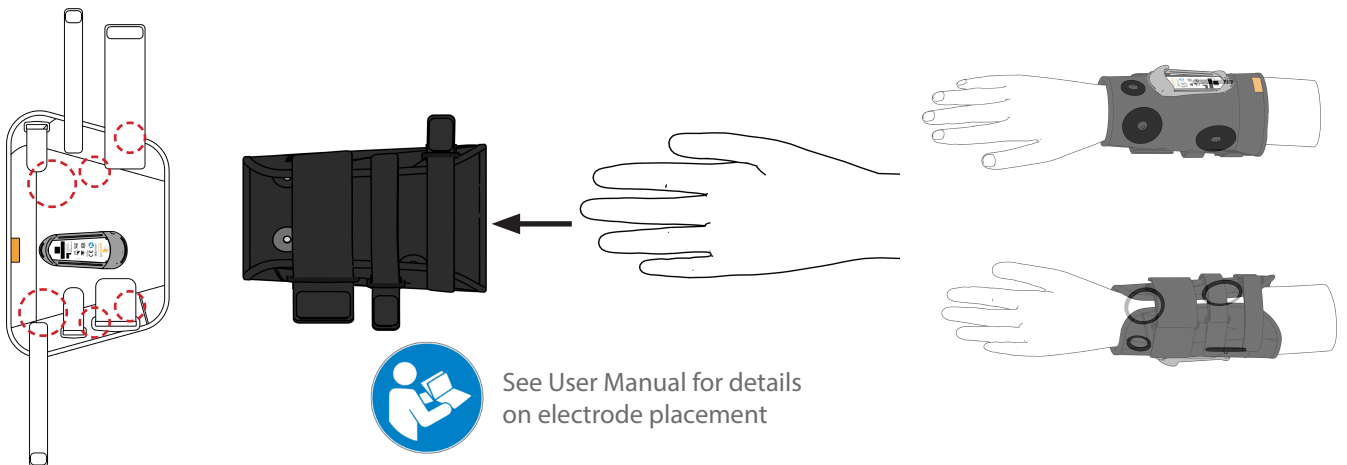

### 4. Connect Stim and Turn On

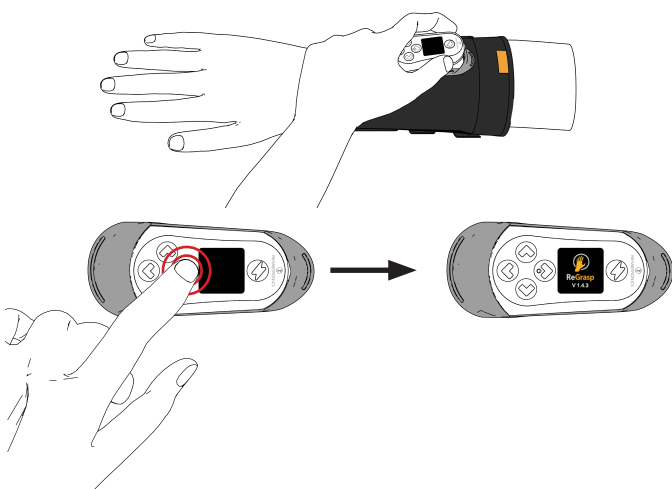


### 5. Enable STIM (⚡)

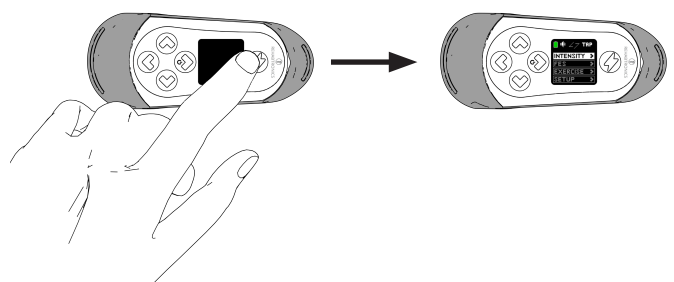





### 6. Set "Hand Open" Intensity

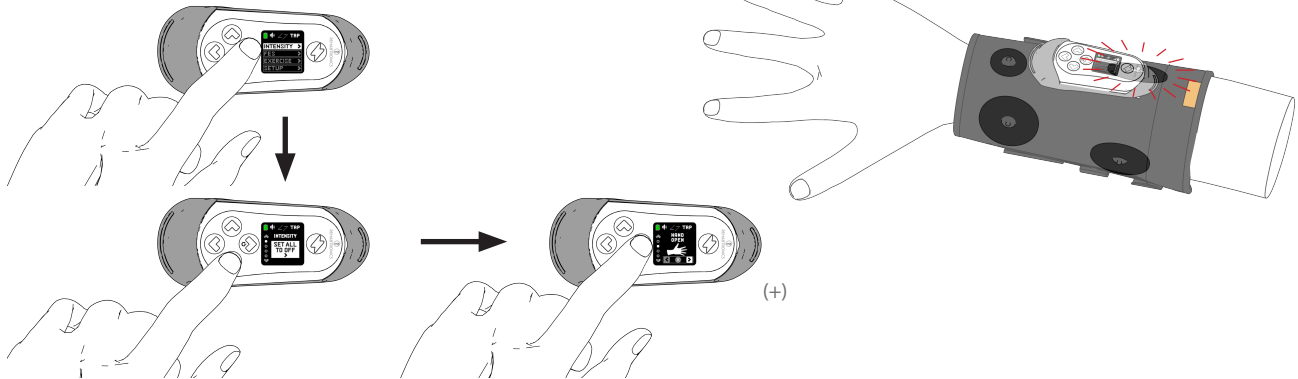

### 7. Set "Hand Close" Intensity

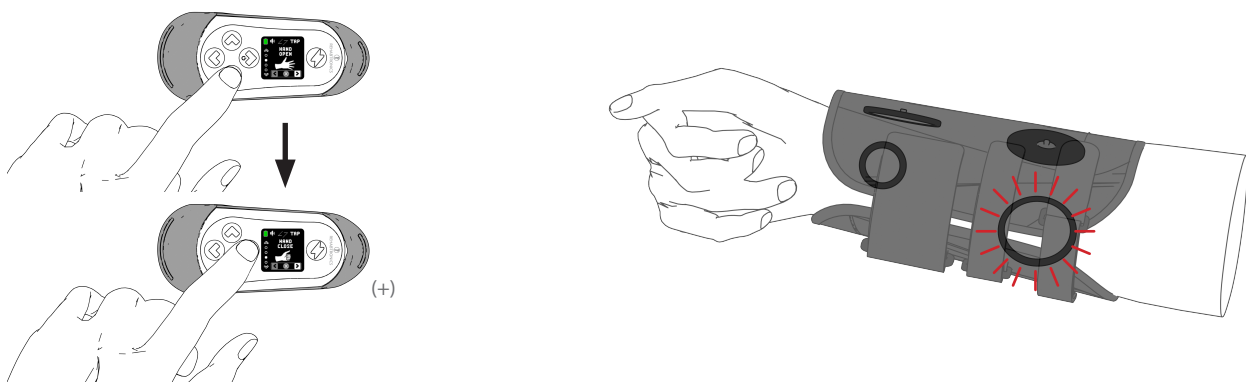

### 8. Initiate "FES" Mode

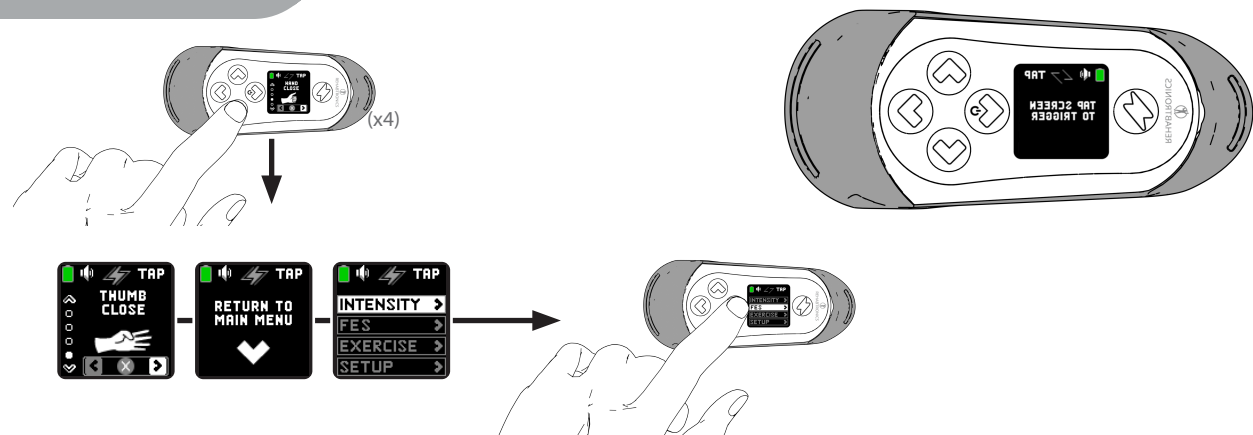

### 9. Tap Stim Display to Cycle Stimulation

